

Nuevo

LOCKnFOLD

Nuevotech's innovative rubber dam dental frame has a lock and fold feature to allow practitioners to take radiographs without removing the rubber dam and frame, preventing tooth contamination during the radiograph procedure.



Health
Canada



Santé
Canada



The LOCKnFOLD rubber dam dental frame has two hinges and two sliding rings. When the sliding rings are moved to the center, the two hinges are unlocked and can be folded. And when they are moved over the hinges, the hinges can be locked. To take radiographs during treatment, the two sliding rings can be moved to the unlocked position and the whole unit can be folded.

As a result, the rubber dam is out of the way, and the practitioner can place the sensor next to the tooth, without removing the rubber dam or the frame. After taking radiographs, the frame can be unfolded back to its original position.

Each hinge can be locked separately. So, one sliding ring can be moved over its hinge to lock it, while the other hinge remains unlocked. This allows the practitioner to work on the locked side and keep the other side folded for suction placement or to help the patient breathe easier. This feature allows the practitioner to maintain tooth isolation and keep the patient comfortable throughout a procedure.

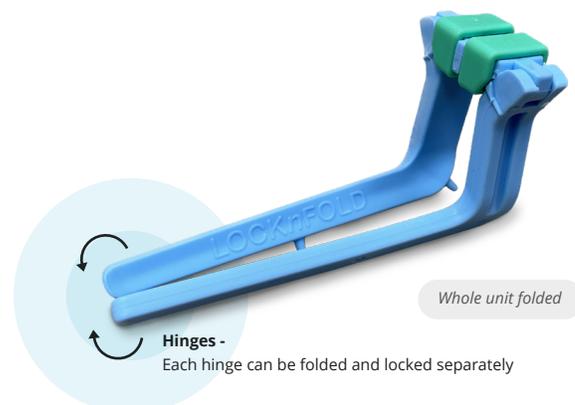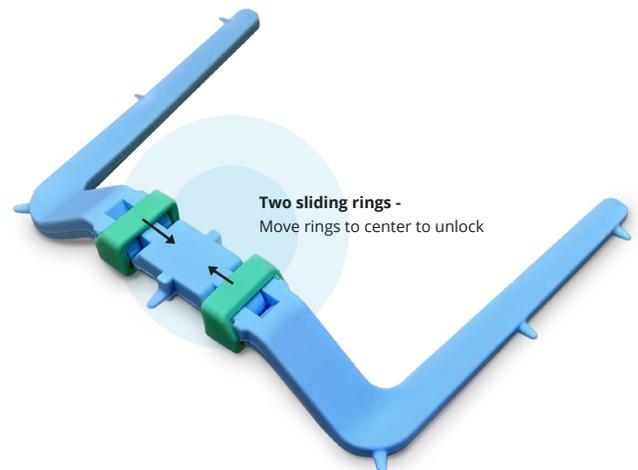

Advantages



Creates a clean field of operation



Enhances vision



Allows radiographs without removing dam



Retracts tissues




Protects soft tissues



Protects the patient



Protects the people from aerosols

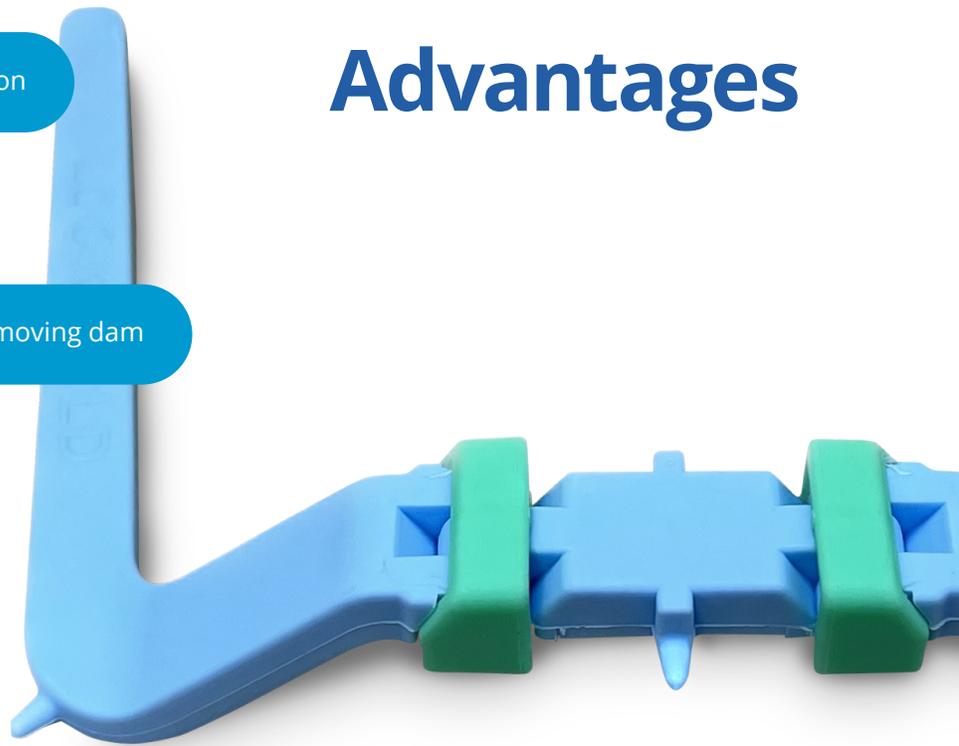

Comparison

	Current	LocknFold
Protects the patient	✓	✓
Easy and quick installation	✗	✓
Comfortable during procedure	✗	✓
It maintains asepsis	✗	✓



Office Address:

Suite 300-1000 3rd Street West
North Vancouver, BC Canada V7P 3J6

Mailing Address:

333-1641 Lonsdale Avenue
North Vancouver, BC Canada V7M 2J5

www.nuevo-tech.com

marketing@nuevo-tech.com
(236) 866-2845